**OXALIKA®**  
made in italy

Via Ugo Foscolo 21/A  
57025 Piombino (LI)  
ITALIA  
info@talitha-info.com  
Tel. +39 370 3432913  
www.talitha-info.com

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\*\*\*\*\*  
\*\*\*\*\*

EN

## Instructions for the Use of : Professional Vaporizer OXALIKA PRO

**Read carefully before using the vaporizer Oxalika Pro. Keep the instructions in a safe place .**

### INDEX

- I. Important Instructions for Safety
- II. Technical details and functions
- III. Treatment and Instructions of Use
- IV. Storage
- V. Maintenance
- VI. Packaging and Product disposal
- VII. Product liability limits
- VIII. Declaration of Conformity

#### I. Important Instructions for Safety

**All safety warnings give specific details of the potential risk present and indicate how to reduce risk of injury, damage and electric shock resulting from improper use of the appliance. Failure to follow these instructions can result in death, fire or electrical shock.**

**The manufacturer declines all responsibility for inappropriate use.**

To reduce the danger of fires, electric shocks, injuries, burns and damage from oxalic acid exposure please comply with the safety guidelines as illustrated below.

1. Read all the instructions before using the device.
2. Use the device only for its intended purpose as described in this manual.
3. Do not use on hives uninhabited by bees.
4. This device is not intended for use by people ( including children ) with reduced physical, sensorial or mental abilities, or those lacking in experience and knowledge, unless they are supervised by a person who is responsible for their safety and ensures the understanding of the dangers caused by improper use.
5. Do not allow children to handle this device.

#### 6. To reduce the dangers of fires and electric shocks:

- Be sure the power voltage specified on the appliance

dataplate is the same as the mains voltage.

**For 12 Volt devices only:**

*The product must be powered by 12 V of direct current. If a battery is used, this must have a minimum capacity of 60 Ah and it must be able to dispense a current of at least 30A continuously. If a power supply is used, this must be able to deliver at least 30 Amperes and have an adequate insulation between input and output. The red and black clamps can be connected randomly at the positive or negative poles. Do not use the product using a damaged battery or power supply which is damaged.*

- Maintenance must be carried out by a qualified technician, in compliance with manufacturer's instructions and local safety regulations.
- Disconnect the power and wait for the appliance to cool down before servicing, repairing and cleaning.
- If the power cable is damaged, request a substitute from the producer, technical assistant or other qualified personnel to avoid danger.
- Grounding is compulsory. Connect appliance to an effective ground (except for the 12 Volt model)
- Do not pull the power supply cord to disconnect it from outlet.
- Do not touch the appliance with any wet part of the body and do not operate it when barefoot.
- Be sure that the electrical cords of other appliances do not touch hot parts of the appliance.
- Do not expose the appliance to atmospheric agents.
- If the appliance, especially the handle, becomes hot due to direct exposure to sunlight, move it to the shade.
- During its use, the electrical power supply system and the OXALIKA vaporizer must be arranged at an adequate distance from flammable materials. Pay special attention to the electric arc that is formed during the linking of the power cables to the poles of the power supply system.
- Do not use the product in case that parts of the device are damaged.

**7. To reduce the risk of burns**

- The dispenser tube [9] the resistance [12] and the thermostat [11] reach temperatures of around 200-220 °C and therefore may cause burns. Do not touch the indicated parts and wear suitable protective gloves to protect against accidental contact.
- The other parts of the vaporizer with the exception of the handle [8], the knobs [1] [6] and the process caps [11] can reach temperatures of about 90-100 °C. The risk of burns is particularly present. Wear suitable protective gloves before touching hot parts.
- Ensure that nobody touches the device when it is hot. After switching off the appliance remains hot for several minutes.
- Do not allow clothes or other flammable materials to come in contact with hot components of the appliance until they have cooled down.

**8. To reduce the risks caused by exposure of oxalic acid:**

- Do not use in closed spaces
- Use only in open spaces in a well ventilated area.
- Do not breathe in vapours or oxalic acid powder.
- Wear a whole body protection suit to avoid contact with vapours or oxalic acid powder, both when filling up the tank and also during treatments.
- Pay special attention to the handling of the oxalic acid.

- Avoid direct contact of the oxalic acid with eyes, skin and respiratory tract.

- Wear safety glasses. Follow the instructions in product designated manual.

- Wear safety gloves. Follow the instructions in user manual.

- Wear a protective mask for respiratory tract with ABE/P3 filter or higher.

Follow the instructions in product user manual.

- Be aware of wind direction during treatment so that the oxalic acid is not blown in the direction of anybody, including the person carrying out the treatment.
- The treatment of bee colonies should happen only outside in open spaces. During and after the treatment ventilate the hive well, nobody should remain in the apiary.
- After the treatment, especially the winter treatment when the bees don't clean like in summer, there may be some oxalic acid crystals inside the hive and on the bottom board. Be careful and wear protective gear.
- Maintain the oxalic acid in its original packaging and in a safe place (with label in accordance with the law)
- As a precaution- wash hands well after use.

**IN CASE OF INGESTION:**

Contact an Antivenom centre or a doctor.

**IN CASE OF CONTACT WITH EYES:**

Wash carefully for a few minutes. Remove contact lenses and keep washing and rinsing.

**IN CASE OF CONTACT WITH SKIN:**

Wash thoroughly with water and soap.

**9. To reduce the risk of injury or work related accidents and illnesses other than those described previously:**

- Do not use the device in dim lighting conditions. In case, use artificial light.
- Clean the device exclusively with water and avoid water entering the handle and coming into contact with the thermic insulation .
- The use of the product by professional operators that use it frequently, must be aware of factors that could increase the risk of danger to the operators themselves.
- Do not leave the device unattended when it is switched on.
- Do not leave the device nor the oxalic acid in places frequented by children and/or where children could reach.
- Handle the device exclusively through the handle[8] , the knobs [1] [6] , the hammer [7] and the caps [2], and the doser caps [10].
- For further information on the use of oxalic acid, please refer to the technical and safety manual of the oxalic acid itself.
- Considering the nature of the device, and its intended use, the dangers that derive specifically with the use of the OXALIKA PRO vaporizer are added to the dangers involved in beekeeping . Faced with these dangers, the professional user will have to remedy after appropriately evaluating them as per applicable laws.

**10. To reduce the risk of damage**

The dispenser tube [9] reaches temperatures of approximately 200 °C and therefore could damage parts of the hive that come into contact with it. Do not use on hives made of plastic or on

plastic nets without adding the designated adapter around the dispenser tube [9].

## II. Technical details and Functions

The oxalic acid is heated so as to make it evaporate when introduced inside the hive. This forms a fog that covers the bees and all the surfaces with a thin layer of oxalic acid crystals. These small crystals are well tolerated by the bees, but have a lethal action on the varroa.

## III. Treatment and Instructions of Use

This device is to be used solely for beekeeping purposes related to the vaporization of the oxalic acid intended for its use.

The vaporizer OXALIKA PRO was designed and built to satisfy specific needs regarding the treatment against varroa of hives inhabited by bees. The vaporizer OXALIKA PRO is used against varroa when there is no brood, during the cold months of the year. Considering the elevated efficacy of the vaporizer OXALIKA PRO (>96%), it is possible to reduce the population of varroa so that the bees are in optimum condition for the next season.

**DOSE:** For the hives, like Dadant, the dose of oxalic acid is 2g for medium colonies. The spoon, Oxalika PRO EASY, corresponds to 1g of acid. If after the treatment more than 1000 varroa are eliminated, or in the uncertainty of brood presence, it is necessary to repeat the treatment at a distance of 2 weeks from the first treatment with the same dose of 1g of acid regardless of the size of the colony. The swarms and artificial swarms can be treated throughout the year. The families with the queen must be treated before the capping of the broods. **Warning:** The orphan colonies react with a frenetic "coming and going" of bees. For further information please see the instructions as provided in the oxalic acid (for beekeeping) information sheet. The device must be used exclusively on hives populated by bees, do not treat weak colonies or empty hives.

In case of any malfunctions and/or damages of parts of the vaporizer OXALIKA PRO stop the use immediately and send it back to the producer for maintenance.

## Product Assembly

Without heating the device, screw the dispenser tube into the threaded hole on the heater and seal it by gripping it with pliers in the zone of greatest diameter.

To facilitate the process of removal and change of dispenser tube there is a lubricating substance on the threaded of the tube.

## Correct use of OXALIKA PRO FAST.

- Ensure the hive doesn't have honey supers.
- Connect the vaporizer OXALIKA PRO (complete with dosing system) to the electric power supply until the point of complete heating (about 8 minutes).
- When the device is heated, fill the tank [3] of the dosing system and close with the top cap. Ease the insertion of the cap by rotating the top knob [1] which is found on the top of the cap [2] in an anticlockwise manner.
- Insert the dispenser tube in the main opening or in a hole made in the rear of an overturned inner cover
- Rotate the top knob found on the cap [2] of the tank in a clockwise manner.
- Make a half turn with the handle [6] connected with the doser [5].

- If the vaporization does not start, use the hammer [7].

EN

- Wait until the vaporization is completed (10-25 seconds) before administering another dose.
- After its use, remove the doser and pour a little bit of water in the heater, letting it flow out of the dispenser tube.
- Avoid water coming into contact with the handle or the insulating material.
- Wash the doser with just water [1,2,3,5,6]

## Correct use of OXALIKA PRO EASY.

- Ensure the hive does not have honey supers.
- Remove the dosing cap from the heater.
- Connect the vaporizer OXALIKA PRO to the electric power supply system until it has completely heated (about 8 minutes).
- Using the dosing spoon insert the oxalic acid into the hole inside the dosing cap.
- Turn over the vaporizer and insert the dosing cap.
- Keeping the vaporizer upside down, insert the dispenser tube in the main opening or in a hole made in the rear of an overturned inner cover.
- Turn the vaporizer upside down and wait for the complete vaporization before removing the cap (about 10-25 seconds)
- Always use two alternating dosing caps
- Do not leave the dosing cap inside the hot vaporizer after the vaporization is completed.
- After use, pour a little bit of water inside the heater and let it flow out of the dispenser tube.
- Avoid water coming into contact with the handle or the insulating material.
- Wash the dosing caps with just water [10].

## IV. Storage

After use, store the vaporizer OXALIKA PRO in a dry place away from direct sunlight.

## V. Maintenance

### Process of Maintenance

- Disconnect the power and wait for the appliance to cool down
- Check the general state of the machine

### Substitution of the fuse inside the handle

N.B.: This job must only be done by an electrical implants professional.

- Disconnect the power and wait for the appliance to cool down
- Loosen the bolts at the end of the handle
- Replace the fuse with an identical one
- Close the device
- In case the device does not work even after replacing the fuse, contact the producer. Absolutely do not try to modify the product.

## VI. Packaging and product disposal

The packaging is made with recyclable material to be disposed of accordingly. The vaporizer OXALIKA PRO, as all electric and electronic devices, must not be disposed of along with domestic waste. Contact the local waste disposal service to obtain information on how to dispose of the product in your own country.

## VII. Limitation of Liability of the Product

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